

## OrthoPets Media

Whether you are (a) sending in your case to get started, (b) fitting your device for the first time, or you are (c) seeking help to solve fit & function concerns; your media provides OrthoPets the data required to provide you with the service and support your pet deserves.

### Camera Positioning Is Critical

To capture the required pictures and videos of your pet, it is critical to:

- Hold your camera at the same elevation as your pet's affected anatomical joint
- Hold your camera stationary while your pet walks through your field of view

OrthoPets has provided a short instructional video demonstrating this technique for you:

- <https://vimeo.com/261067863>

### 3-2-1: Media Process


Using our simple 3-2-1 media process, you can quickly and effortlessly provide OrthoPets with the required media we need to assess your pet's conformation, angulation, and gait pattern.

**3 Still Photos** of your pet's affected limb while your pet is standing:

- Front (cranial) surface of limb
- Outside (lateral) surface of limb
- Back (caudal) surface of limb

**2 Walking Videos** of your pet:

- Side-to-side:
  - Your pet walks from left to right and right to left in front of the camera
- Front-to-back:
  - Your pet walks towards the camera and away from the camera

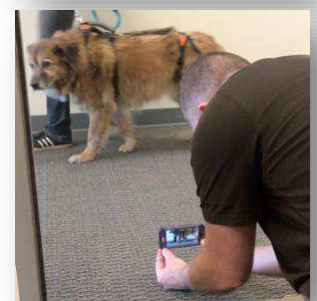

**1 Simple Way** to send media:

- OrthoPets utilizes "Filemail" transfer program enabling you to send large files from your mobile device, tablet, or desktop computer
  - <https://orthopets.com/my-pets-journey/>