


Toe-Up Device with Boot - Measurement Instructions

For both measurements, please use a fabric tape measurer.

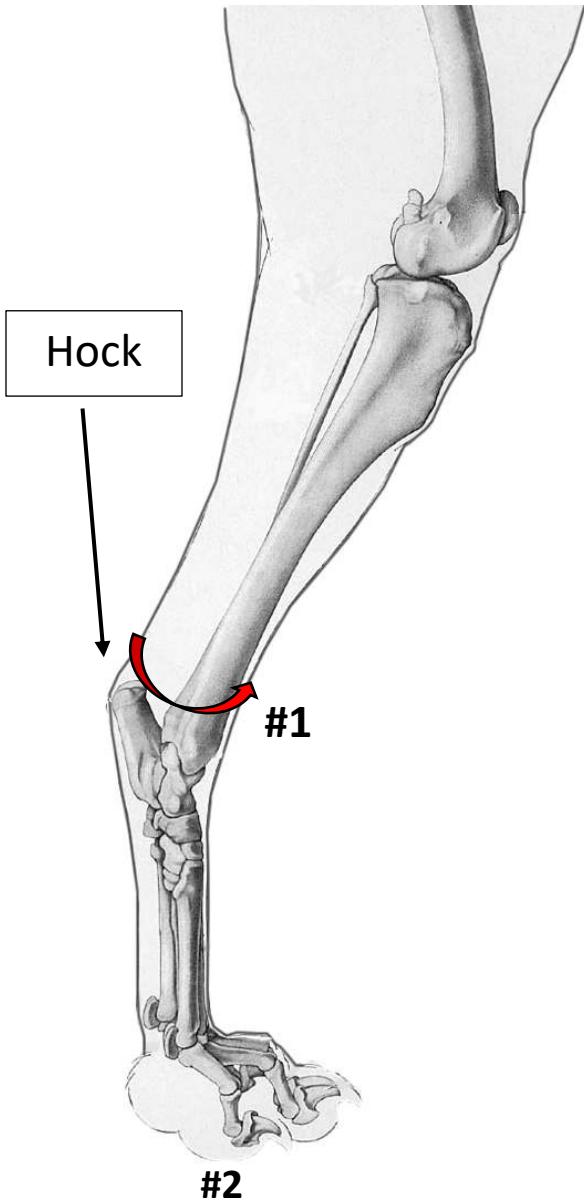

#1 Tarsal circumference (just above point of hock):

- Measure the circumference around the tarsus/hock joint (just above the bulbous part of the back of the hock).
- This measurement will correspond with the Toe-Up size (portion that wraps around the hock).

#2 Paw width measurement (across bottom aspect of paw):

- Record the width of the paw at the widest point.
- Take this measurement while the pet is standing/weight bearing.
- This measurement will correspond with the boot size.

